
PIEDRO®

always onestep ahead

Piedro Semi-Orthopedic Footwear

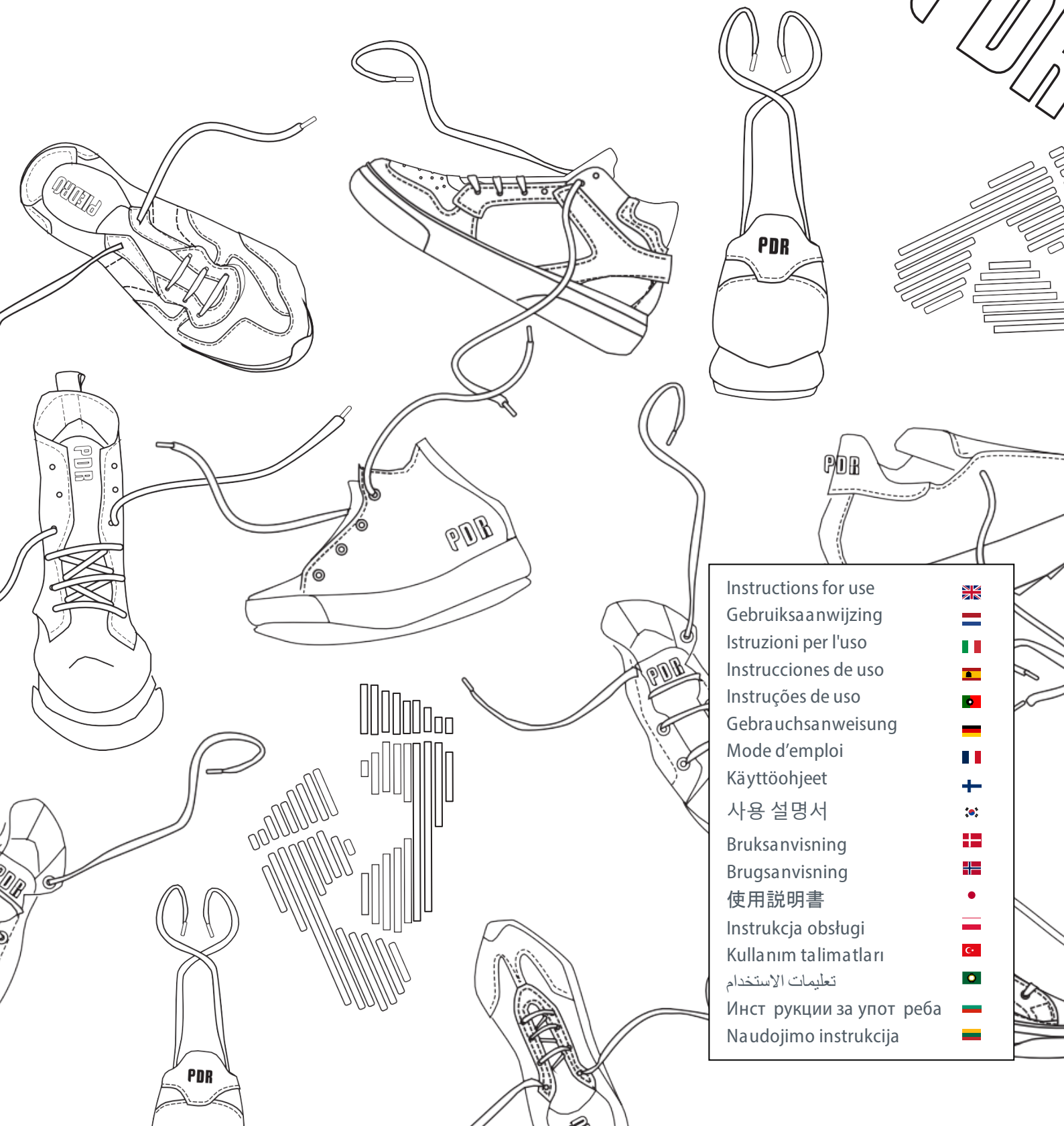Instructions for use

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Mode d'emploi

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Bruksanvisning

Brugsanvisning

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1. Instructions for use

Please read the instructions carefully before using our semi-orthopedic shoes.

1.1 Intended Use

Piedro delivers semi-orthopedic shoes which are designed to provide some level of orthopedic support and comfort. The intended use for semi-orthopedic shoes includes:

1. Daily Wear
2. Work and Office Settings
3. Casual and Social Occasions
4. Mild Foot Conditions, when custom made footwear is not needed yet.
5. Travel
6. Recovery
7. Preventive Use

It's important to note that while semi-orthopedic shoes provide more support and comfort than regular fashion shoes, they will only be delivered after consulting a healthcare specialist. Semi-orthopedic footwear is typically recommended for individuals who need more support, but don't need custom-made orthopedic shoes. The fitting of the shoes must always be done by highly qualified personnel.

1.2 Indications for Use

The Piedro semi-orthopedic diabetic shoes is recommended for use by individuals experiencing any of the following conditions:

1. Foot Discomfort (neuropath, PAD)
2. Mild Foot Conditions
3. Recovery from Minor Foot Injuries
4. Preventive Footcare
5. Diabetic Feet
6. Rheumatic Affected Feet
7. Sensitive Feet
8. Aging Feet
9. Wearing individual inlays.

When you have diabetes take extra care.

The Piedro semi-orthopedic AFO shoes is recommended for use by individuals experiencing any of the following conditions:

1. When wearing an Ankle Foot Orthoses (AFO).
2. Individuals who require an AFO to address various medical affecting the lower limbs.

The Piedro semi-orthopedic stability and rehabilitation shoes is recommended for use by individuals experiencing any of the following conditions:

1. Specific pronation issues
2. Overpronation
3. Recovery from Minor Foot Injuries
4. Preventive Footcare
5. Extreme varus or valgus instability
6. When a foot needs more support
7. Cerebral Palsy (CP)
8. Individual inlays

Your health care professional will indicate whether you need a rehabilitation shoe or the even more supportive stability shoe.

The Piedro semi-orthopedic reverse last shoes is recommended for use by individuals experiencing any of the following conditions:

1. Club feet
2. Extreme varus and or adduction of the forefoot
3. Pes equino-varus
4. Pes varus
5. Post-operative surgery support

You can wear Piedro semi-orthopedic footwear every day during travel, work but also during casual and dressy occasions. It's important to note that while semi-orthopedic shoes provide more support and comfort than regular fashion shoes, they may not be suitable for individuals with severe or specific foot that require custom orthotics or specialized orthopedic shoes. In such cases, consulting a healthcare provider or podiatrist is advisable to ensure proper footwear selection and fit.

1.3 Contraindications

The semi-orthopedic shoes must not be used when having:

1. Severe Foot Deformities
2. Recent Foot Surgery
3. Specific Medical Conditions
4. Severe Gait Abnormalities
5. Allergies or Sensitivities
6. when it is not the correct size.

1.4 Symbols Glossary

Consult instructions for use

Manufacturer

Caution

CE Logo

UKCA Marking

1.5 General Instructions

The Instructions for use must be read and understood before attempting to use the shoes.

Piedro shoes are suitable for both indoor and outdoor use.

Piedro shoes must be provided by qualified personnel. The duration and frequency of use are determined by qualified personnel.

Putting on and taking off the Piedro shoes will be easier with a relative/carer assisting.

Always use a shoe horn when putting on the shoes to prevent wrinkles in the heel lining.

Make sure that the shoes are properly secured to avoid the risk of falling.

All persons involved in the fitting and maintenance of the shoes must read and understand the Instructions for Use prior to using the Piedro shoes.

Care should be taken to instruct the user and carer on the correct fitting procedure as shown under "Fitting Instructions".

Piedro semi-orthopedic shoes is a medical product and is intended exclusively for one patient. Sharing and use by others is not allowed.

During the initial fitting of the Piedro shoes ensure that the Piedro shoes are fitted securely enough to provide support, but not lead to discomfort or unexpected issues. Wear new shoes only for short intervals and check your feet for pressure points after 20 minutes.

Check every day before putting on the Piedro shoes to make sure there are no foreign object inside the shoes.

If any pressure points can be felt, adjust/reposition the foot slightly to alleviate the load.

Check your feet daily for redness, pressure points, or changes in the skin. Contact your medical team immediately if necessary.

If the Piedro shoes should cause any problems or pain. Consult your medical team if this occurs.

Do not use if there is significant damage/ deterioration of the Piedro shoes.

If in doubt, please refer to your medical team treating you for advice before using the Piedro shoes.

If users have sensitive skin, a small proportion may experience minor irritation from the Piedro shoes material. Prior to fitting the Piedro shoes, the Piedro shoes should be fitted to the user for a short period first, then removed and a visual inspection of the area made to ensure no skin reaction has taken place.

In the unlikely event that if while wearing the Piedro shoes, that any of the following issues are experienced: Discomfort / skin irritation – the Piedro shoes must be removed. If symptoms do not resolve swiftly after its removal, contact your medical team treating you for advice. If any continued symptoms are experienced following use of the Piedro shoes, please contact your medical team treating you.

1.6 Fitting Instructions

Putting on and taking off the Piedro shoes will be easier with an assistant, please ask them to read the instructions for use in full first.

Piedro shoes must be fitted by an appropriately trained medical professional.

1.7 When to Consult a Healthcare Professional

Piedro shoes should be removed if any of the following issues are experienced:

- Discomfort
- Skin irritation

If any continued adverse symptoms are experienced following use of Piedro shoes, please contact the medical team treating you. Ensure that you have read in full the instructions for use. If in doubt, please refer to your medical team treating you.

1.8 Care and Cleaning Instructions

Do not use if any damage is apparent.

Refer to the Fitting Instructions after maintenance of Piedro shoes.

Do not machine wash the Piedro shoes.
Do not use bleach. Do not tumble dry.
Do not iron any part of the Piedro shoes.

Proper care and maintenance of Piedro shoes are essential to ensure their longevity and effectiveness in providing support and comfort for your feet. Follow these care and cleaning instructions to keep your shoes in the best possible condition.

Regular Inspection: Before and after each wear, inspect your orthopedic shoes for any signs of damage or wear. Check for loose stitching, worn-out soles, or any other issues that may affect their functionality.

Storage: Store your Piedro shoes in a cool, dry place away from direct sunlight, extreme heat, or cold temperatures. Use shoe trees or shoe shapers to help maintain their shape and prevent creasing.

Cleaning: Use a soft, damp cloth or sponge to gently wipe away surface dirt and stains. Avoid using abrasive materials or harsh chemicals that could damage the shoe's material.

For leather or synthetic uppers, use a specialized leather cleaner or conditioner to keep the material supple and clean. Take the removable insoles out and clean them separately according to the manufacturer's instructions.

Drying: Never expose Piedro shoes to direct heat sources such as radiators or hairdryers, as this can cause the materials to crack or shrink. After wearing your shoes in wet conditions, stuff them with newspaper to help absorb moisture and maintain their shape. Allow them to air dry naturally at room temperature.

Odor Control: To prevent and reduce odors, ask your health carer to make insoles from odor-absorbing insoles or inserts, and allow your shoes to air out between wears. Consider using shoe deodorizers or sprays designed for this purpose.

Protection: Apply a waterproofing spray to your shoes if they are not already waterproofed. This will help protect them from water and stains.

Lacing and Fasteners: Check the laces, straps, or other fasteners regularly for signs of wear and tear. Replace them if necessary to maintain proper support and fit.

Rotation: If you wear your Piedro orthopedic shoes daily, consider rotating them with another pair to allow each pair to air out and prevent excessive wear on one set.

By following these care and cleaning instructions, you can extend the life of your semi orthopedic shoes and continue to enjoy the support and comfort they provide for your feet. Remember that proper maintenance is essential for ensuring their effectiveness in promoting foot health and overall well-being.

1.9 Disposal

The disposal and recycling of used shoes and packaging must comply with the applicable legal regulation in each country.

1.10 Terms and Conditions

For terms and conditions please visit www.piedro.com

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Piedro International B.V.